

# **Test Definition: VITE**

Vitamin E, Serum

**Reporting Title:** Vitamin E, S **Performing Location:** Rochester

#### **Shipping Instructions:**

Ship specimen in amber vial to protect from light.

# **Specimen Requirements:**

Patient Preparation: Patient should fast overnight (12-14 hours); infants should have specimen collected before next

feeding

Supplies: Amber Frosted Tube, 5 mL (T915)

**Collection Container/Tube:** 

**Preferred:** Red top **Acceptable:** Serum gel

Submission Container/Tube: Amber vial

Specimen Volume: 0.5 mL

Collection Instructions: Centrifuge and aliquot serum into light protected plastic vial within 2 hours of collection.

#### Forms:

If not ordering electronically, complete, print, and send a General Request (T239) with the specimen.

Specimen Type	Temperature	Time	Special Container	
Serum	Ambient	7 days	LIGHT PROTECTED	
	Refrigerated (preferred)	44 days	LIGHT PROTECTED	
	Frozen	44 days	LIGHT PROTECTED	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
2350	A-Tocopherol, Vitamin E	Numeric	mg/L	1823-4

LOINC® and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

### **CPT Code Information:**

84446

#### **Reference Values:**

0-17 years: 3.8-18.4 mg/L > or =18 years: 5.5-17.0 mg/L