

Prolactin, Serum

Reporting Title: Prolactin, S **Performing Location:** Rochester

Specimen Requirements:

Patient Preparation: For 12 hours before specimen collection do not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.
Container/Tube:
Preferred: Serum gel
Acceptable: Red top
Submission Container/Tube: Plastic vial
Specimen Volume: 0.6 mL
Collection Instructions: Centrifuge and aliquot serum into a plastic vial.

Forms:

If not ordering electronically, complete, print, and send an <u>Oncology Test Request</u> (T729) with the specimen.

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Ambient | 24 hours | |
| | Refrigerated (preferred) | 7 days | |
| | Frozen | 90 days | |

Result Codes:

| Result ID | Reporting Name | Туре | Unit | LOINC® |
|-----------|----------------|---------|-------|---------|
| PRL | Prolactin, S | Numeric | ng/mL | 20568-2 |

LOINC[®] and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84146

Reference Values:

Males <18 years: not established > or =18 years: 4.0-15.2 ng/mL

Females: <18 years: not established > or =18 years: 4.8-23.3 ng/mL

For SI unit Reference Values, see https://www.mayocliniclabs.com/order-tests/si-unit-conversion.html



Test Definition: PRL

Prolactin, Serum