

# **Test Definition: ADULT**

Adulterants Survey, Random, Urine

## Overview

### **Useful For**

Assessment of possible adulteration of a urine specimen submitted for drug of abuse testing

Providing the creatinine concentration for normalization purposes

### **Testing Algorithm**

For more information see Adulterant Survey Algorithm.

#### **Special Instructions**

<u>Adulterant Survey Algorithm</u>

Method Name Spectrophotometry (SP)

NY State Available Yes

#### Specimen

Specimen Type Urine

#### **Ordering Guidance**

For situations where chain of custody is required, a Chain-of-Custody Kit (T282) is available. For chain-of-custody information, see ADLTX / Adulterants Survey, Chain of Custody, Random, Urine.

#### Specimen Required

Container/Tube: Plastic, 60-mL urine bottle
Specimen Volume: 1.5 mL
Collection Instructions:

Collect a random urine specimen.
No preservative.

Additional Information: Submitting less than 1.5 mL may compromise the ability to perform all necessary testing.

### Forms

If not ordering electronically, complete, print, and send a <u>Therapeutics Test Request</u> (T831) with the specimen.

#### Specimen Minimum Volume

1 mL



Adulterants Survey, Random, Urine

# **Reject Due To**

All specimens will be evaluated at Mayo Clinic Laboratories for test suitability.

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Urine	Ambient	72 hours	
	Refrigerated (preferred)	14 days	
	Frozen	14 days	

# **Clinical & Interpretive**

## **Clinical Information**

Specimen adulteration is the manipulation of a sample that may cause false-negative test results for the presence of drugs of abuse. Common adulterants that may affect testing are water, soap, bleach, vinegar, oxidants, and salt. The adulteration testing includes assessment of creatinine concentration, pH, urine specific gravity, presence or absence of an oxidant, and presence or absence of nitrite.

# **Reference Values**

Cutoff concentrations Oxidants: 200 mg/L Nitrites: 500 mg/L

## Interpretation

For information see Adulterant Survey Algorithm.

## Cautions

No significant cautionary statements

## **Clinical Reference**

1. US Department of Health and Human Services, Substance Abuse and Mental Health Services Administration (SAMHSA). Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register. January 23, 2017;82(13):FR 7920. Accessed December 13, 2024. Available at

www.samhsa.gov/sites/default/files/workplace/frn\_vol\_82\_7920\_.pdf

2. US Department of Health and Human Services, Substance Abuse and Mental Health Services Administration (SAMHSA). Drug-Free Workplace Programs: Employer Resources. Updated July 23, 2024. Accessed December 13, 2024. Available at www.samhsa.gov/workplace/resources

3. US Department of Health and Human Services (HHS), Substance Abuse and Mental Health Services Administration (SAMHSA). Mandatory Guidelines for Federal Workplace Drug Testing Programs. Updated October 12, 2023. Accessed December 13, 2024. Available at

www.federalregister.gov/documents/2023/10/12/2023-21734/mandatory-guidelines-for-federal-workplace-drug-testin g-programs



# **Test Definition: ADULT**

Adulterants Survey, Random, Urine

# Performance

## **Method Description**

All results are measured using spectrophotometry at wavelengths specified by the reagent manufacturer. The use of a refractometer may also be used in the specific gravity measurement. (Package inserts: Specimen Validity Test Creatinine. Roche Diagnostics; V3.0, 08/2015; Specimen Validity Test Nitrite. Roche Diagnostics; V3.0, 08/2018, Specimen Validity Test Oxidant. Roche Diagnostics; V 3.0, 08/2018; Specimen Validity Test pH Roche Diagnostics; V3.0, 02/2019, Specimen Validity Test Specific Gravity. Roche Diagnostics; V4.0, 08/2022)

PDF Report

No

Day(s) Performed Monday through Saturday

**Report Available** Same day/1 to 2 days

Specimen Retention Time 2 weeks

Performing Laboratory Location Rochester

# Fees & Codes

#### Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact Customer Service.

## **Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

## LOINC<sup>®</sup> Information

Test ID	Test Order Name	Order LOINC <sup>®</sup> Value
ADULT	Adulterants Survey, U	58714-7
Result ID	Test Result Name	Result LOINC <sup>®</sup> Value
20606	Creatinine, U	2161-8



# **Test Definition: ADULT**

Adulterants Survey, Random, Urine

22312	Specific Gravity	In Process
23509	рН	2756-5
23511	Oxidants	58714-7
23510	Nitrites	32710-6
30914	Comment	48767-8