

**Overview****Method Name**

ImmunoCAP FEIA

**NY State Available**

Yes

**Specimen****Specimen Type**

Serum

**Specimen Required****Specimen Type:** Serum**Container/Tube:** Red or SST**Specimen Volume:** 0.5 mL**Collection Instructions:** Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.**Specimen Minimum Volume**

0.5 mL

**Reject Due To**

|          |                  |
|----------|------------------|
| Thawing: | Warm OK; Cold OK |
| Lipemia: | Reject           |

**Specimen Stability Information**

| Specimen Type | Temperature              | Time     | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum         | Refrigerated (preferred) | 28 days  |                   |
|               | Frozen                   | 365 days |                   |
|               | Ambient                  | 28 days  |                   |

**Clinical & Interpretive****Reference Values**

&lt;0.35 kU/L

**Interpretation**

| Class | IgE (kU/L)  | Comment              |
|-------|-------------|----------------------|
| 0     | <0.10       | Negative             |
| 0/1   | 0.1-0.34    | Equivocal/Borderline |
| 1     | 0.35-0.69   | Low Positive         |
| 2     | 0.70-3.49   | Moderate Positive    |
| 3     | 3.50-17.49  | High Positive        |
| 4     | 17.50-49.99 | Very High Positive   |
| 5     | 50.00-99.99 | Very High Positive   |
| 6     | >99.99      | Very High Positive   |

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

3 to 6 days

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

86003

**LOINC® Information**

---

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
| FCOFE   | Coffee IgE      | 6083-0             |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3084     | Coffee IgE       | 6083-0              |
| Z3085     | Class            | 102248-2            |