

## Overview

### Method Name

Enzyme Immunoassay (EIA)

### NY State Available

Yes

## Specimen

### Specimen Type

Serum

### Specimen Required

**Specimen Type:** Serum

**Container/Tube:** Red Top

**Specimen Volume:** 0.5 mL

**Collection Instructions:** Draw blood in a plain re-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

### Specimen Minimum Volume

0.5 mL

### Reject Due To

|          |                  |
|----------|------------------|
| Thawing: | Cold OK; Warm OK |
| Lipemia: | Reject           |

### Specimen Stability Information

| Specimen Type | Temperature              | Time     | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum         | Refrigerated (preferred) | 28 days  |                   |
|               | Frozen                   | 365 days |                   |
|               | Ambient                  | 28 days  |                   |

## Clinical & Interpretive

### Reference Values

<0.35 kU/L

## Interpretation

| <a href="#">Class</a> | <a href="#">IgE (kU/L)</a> | <a href="#">Comment</a> |
|-----------------------|----------------------------|-------------------------|
| 0                     | <0.35                      | Below Detection         |
| 1                     | 0.35-0.69                  | Low Positive            |
| 2                     | 0.70-3.49                  | Moderate Positive       |
| 3                     | 3.50-17.49                 | Positive                |
| 4                     | 17.50-49.99                | Strong Positive         |
| 5                     | >49.99                     | Very Strong Positive    |

## Performance

### Method Description

### PDF Report

No

### Day(s) Performed

Monday through Friday

### Report Available

4 to 7 days

### Performing Laboratory Location

Eurofins Viracor

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### Test Classification

[This test was developed and its performance characteristics determined by Eurofins Viracor . It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

### CPT Code Information

86003

---

**LOINC® Information**

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
| FSCNE   | Sugar Cane IgE  | 7723-0             |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3102     | Sugar Cane IgE   | 7723-0              |
| Z3103     | Class            | 102615-2            |