

Overview

Method Name
High Performance Liquid Chromatography with Ultraviolet Detection; (HPLC-UV)

NY State Available
Yes

Specimen

Specimen Type
Varies

Specimen Required
Submit only 1 of the following specimens:

Plasma
Draw blood in a green-top sodium heparin tube(s), plasma gel tube is not acceptable. Spin down and send 3mL of plasma refrigerated in a plastic vial.

Serum
Draw blood in a plain, red-top tube(s), serum gel tube is not acceptable. Spin down and send 3 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume
0.6 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Ambient	72 hours	
	Refrigerated (preferred)	7 days	
	Frozen	180 days	

Clinical & Interpretive

Reference Values

Units: ng/mL

Expected fluvoxamine concentrations on recommended daily dosage regimens:
50-900 ng/mL

Performance

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

3 to 7 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80332

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FFVOX	Fluvoxamine (Luvox)	10988-4

Result ID	Test Result Name	Result LOINC® Value
FFVOX	Fluvoxamine (Luvox)	10988-4