

# **Test Definition: FUNBW**

Fungitell, bronch wash

#### Overview

#### Method Name

Limulus Amebocyte Lysate (LAL) Pathway

#### NY State Available

Yes

Specimen

## Specimen Type

**Bronchial Washing** 

#### Specimen Required

Collect 1-3 mL Bronchial Wash in a sterile screw top tube, ship frozen.

#### **Specimen Minimum Volume**

0.5 mL

#### Reject Due To

Hemolysis	Mild reject; Gross reject	
Lipemia	Mild reject; Gross reject	
Icterus	Mild reject; Gross reject	
Other	Non screw top tube	

#### **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Bronchial Washing	Ambient	4 days	
	Refrigerated	7 days	
	Frozen (preferred)	30 days	

#### Clinical & Interpretive

#### **Clinical Information**

The Fungitell Beta – D Glucan assay detects (1,3) – Beta-D-glucan from the following pathogens: Candida spp., Acremonium, Aspergillus spp., Coccidioides immitis, Fusarium spp., Histoplasma capsulatum, Trichosporon spp., Sporothrix schenckii, Saccharomyces cerevisiae, and Pneumocystis jiroveci.



The Fungitell Beta-D Glucan assay does not detect certain fungal species such as the genus Cryptococcus, which produces very low levels of (1,3) – Beta-D-glucan, nor the Zygomycetes, such as Absidia, Mucor, and Rhizopus, which are not known to produce (1,3) – Beta-D-glucan. Studies indicate Blastomyces dermatitidis is usually not detected due to little (1,3) – Beta-D-glucan produced in the yeast phase.

### **Reference Values**

A reference range for specimens other than serum has not been established.

#### Interpretation

The performance characteristics of the Fungitell assay in bronchial wash have been determined by Viracor-IBT Laboratories; there are no established criteria for the interpretation of Fungitell results from bronchial wash fluid. Research studies have evaluated the use of the Fungitell assay in BAL in both immunocompromised patients (Mycopathologia (2013) 175:33-41) and acute eosinophilic pneumonia (Chest (2013) 123:1302-1307).

# Performance

# PDF Report

Day(s) Performed Monday through Saturday

Report Available

1 to 6 days

**Performing Laboratory Location** Eurofins Viracor

#### Fees & Codes

#### Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **CPT Code Information**

87449

#### LOINC<sup>®</sup> Information

Test ID Tes	est Order Name	Order LOINC <sup>®</sup> Value
-------------	----------------	--------------------------------



MAYO CLINIC LABORATORIES

Fungitell, bronch wash

FUNBW	Fungitell, bronch wash	Not Provided
Result ID	Test Result Name	Result LOINC <sup>®</sup> Value
FUNBW	Fungitell, bronch wash	Not Provided