

Overview

Method Name

Limulus Amebocyte Lysate (LAL) Pathway

NY State Available

Yes

Specimen

Specimen Type

Lavage

Specimen Required

Specimen Type: BAL

Sources: Bronchoalveolar lavage

Container/Tube: Sterile container

Specimen Volume: 1 – 3 mL

Note: Ship frozen

Specimen Minimum Volume

0.5 mL

Reject Due To

Hemolysis	Mild reject; Gross reject
Lipemia	Mild reject; Gross reject
Icterus	Mild reject; Gross reject
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Lavage	Frozen (preferred)	30 days	
	Refrigerated	7 days	
	Ambient	4 days	

Clinical & Interpretive

Clinical Information

The Fungitell Beta β -D Glucan assay detects (1,3) β -D-glucan from the following pathogens: *Candida* spp., *Acremonium*, *Aspergillus* spp., *Coccidioides immitis*, *Fusarium* spp., *Histoplasma capsulatum*, *Trichosporon* spp., *Sporothrix schenckii*, *Saccharomyces cerevisiae*, and *Pneumocystis jiroveci*.

The Fungitell Beta-D Glucan assay does not detect certain fungal species such as the genus *Cryptococcus*, which produces very low levels of (1,3) β -D-glucan, nor the Zygomycetes, such as *Absidia*, *Mucor*, and *Rhizopus*, which are not known to produce (1,3) β -D-glucan. Studies indicate *Blastomyces dermatitidis* is usually not detected due to little (1,3) β -D-glucan produced in the yeast phase.

Reference Values

A reference range for specimens other than serum has not been established.

Interpretation

The performance characteristics of the Fungitell assay in BAL have been determined by Viracor-Eurofins; there are no established criteria for the interpretation of Fungitell results from BAL fluid. Research studies have evaluated the use of the Fungitell assay in BAL in both immunocompromised patients (*Mycopathologia* (2013) 175:33-41) and acute eosinophilic pneumonia (*Chest* (2003) 123:1302-1307).

Performance**PDF Report**

No

Day(s) Performed

Monday through Saturday

Report Available

1 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

87449

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FUNBL	Fungitell, BAL	Not Provided

Result ID	Test Result Name	Result LOINC® Value
FUNBL	Fungitell, BAL	Not Provided