

Overview

Method Name  
Enzyme Immunoassay (EIA)

NY State Available  
Yes

Specimen

Specimen Type  
Serum

Specimen Required  
Specimen Type: Serum  
Container/Tube: Red or SST  
Specimen Volume: 0.5 mL  
Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume  
0.5 mL

Reject Due To

Thawing:	Warm OK; Cold OK
Lipemia:	Reject

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Ambient	28 days	
	Refrigerated (preferred)	28 days	
	Frozen	365 days	

Clinical & Interpretive

Reference Values  
<0.35 kU/L

Interpretation

<a href="#">Class</a>	<a href="#">IgE (kU/L)</a>	<a href="#">Comment</a>
0	<0.35	Below Detection
1	0.35-0.69	Low Positive
2	0.70-3.49	Moderate Positive
3	3.50-17.49	Positive
4	17.50-49.99	Strong Positive
5	>49.99	Very Strong Positive

Performance

Method Description

The conventional EIA uses allergen-coated discs from several suppliers and an enzyme-labeled anti-IgE.

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

2 to 3 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FYOG	Yogurt IgE	7787-5

Result ID	Test Result Name	Result LOINC® Value
Z3868	Yogurt IgE	7787-5
Z3869	CLASS	102848-9