

Overview

Method Name

ImmunoCAP FEIA

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Specimen Type: Serum

Container/Tube: Red or SST

Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

| | |
|---------------|--------|
| Gross lipemia | Reject |
|---------------|--------|

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Ambient | 28 days | |
| | Refrigerated (preferred) | 28 days | |
| | Frozen | 365 days | |

Clinical & Interpretive

Reference Values

<0.35 kU/L

Interpretation

| Class | IgE (kU/L) | Comment |
|-------|---------------|----------------------|
| 0 | <0.10 | Negative |
| 0/1 | 0.10 – 0.34 | Equivocal/Borderline |
| 1 | 0.35 – 0.69 | Low Positive |
| 2 | 0.70 – 3.49 | Moderate Positive |
| 3 | 3.50 – 17.49 | High Positive |
| 4 | 17.50 – 49.99 | Very High Positive |
| 5 | 50.00 – 99.99 | Very High Positive |
| 6 | >99.99 | Very High Positive |

Performance

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

3 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

[This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

CPT Code Information

86003

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
|---------|-----------------|--------------------|

| | | |
|-----------|------------------|---------------------|
| FMME | Mahi Mahi IgE | 82570-3 |
| Result ID | Test Result Name | Result LOINC® Value |
| Z3886 | Mahi Mahi IgE | 82570-3 |
| Z3887 | CLASS | 103121-0 |