

Overview

Method Name

Gel Diffusion

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Specimen Type: Serum

Container/Tube: Red or SST

Specimen Volume: 1 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 1 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

| | |
|-----------|--------|
| Hemolysis | Reject |
| Lipemia | Reject |
| Icterus | Reject |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum | Ambient | 28 days | |
| | Refrigerated (preferred) | 28 days | |
| | Frozen | 28 days | |

Clinical & Interpretive

Reference Values

Negative

Interpretation

The gel diffusion method was used to test this patient's serum for the presence of precipitating antibodies (IgG) to the antigens indicated. These antibodies are serological markers for exposure and immunological sensitization. The clinical significance varies, depending on the history and symptoms.

Performance

PDF Report

No

Day(s) Performed

Tuesday, Friday

Report Available

7 to 11 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86331 x 10

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------------------------|--------------------|
| FBFPI | Bird Fancier's Precipitin Panel I | Not Provided |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|----------------------------------|---------------------|
| Z3892 | Parakeet Droppings Gel Diffusion | 24503-5 |

| | | |
|-------|-------------------------------------|--------------|
| Z3894 | Parrott Droppings Gel Diffusion | 24501-9 |
| Z3890 | Cockatiel Droppings Gel Diffusion | 23843-6 |
| Z3891 | Finch Droppings Gel Diffusion | Not Provided |
| Z3893 | Parakeet Serum Gel Diffusion | 24502-7 |
| Z3888 | Canary Droppings Gel Diffusion | Not Provided |
| Z3889 | Chicken Serum Gel Diffusion | 24498-8 |
| Z3895 | Parrott Serum Gel Diffusion | 24500-1 |
| Z3897 | Pigeon/Dove Serum Gel Diffusion | 6733-0 |
| Z3896 | Pigeon/Dove Droppings Gel Diffusion | 24505-0 |