

Overview**Method Name**

ImmunoCAP FEIA

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required**Specimen Type:** Serum**Container/Tube:** Red Top**Specimen Volume:** 0.5 mL**Collection Instructions:** Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.**Specimen Minimum Volume**

0.5 mL

Reject Due To

Lipemia	Reject
---------	--------

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	28 days	

Clinical & Interpretive**Reference Values**

<0.35 kU/L

Interpretation

<u>Class</u>	<u>IgE (kU/L)</u>	<u>Comment</u>
0	<0.10	Negative
0/1	0.10 â€” 0.34	Equivocal/Borderline
1	0.35 â€” 0.69	Low Positive
2	0.70 â€” 3.49	Moderate Positive
3	3.50 â€” 17.49	High Positive
4	17.50 â€” 49.99	Very High Positive
5	50.00 â€” 99.99	Very High Positive
6	>99.99	Very High Positive

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

3 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

[This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

CPT Code Information

86003

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
---------	-----------------	--------------------

FOAKE	Oak Live (Quercus virginiana) IgE	6164-8
-------	-----------------------------------	--------

Result ID	Test Result Name	Result LOINC® Value
Z4118	Oak Live (Quercus virginiana) IgE	6164-8
Z4119	CLASS	102469-4