

Overview

Method Name

Hemolysis

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

Specimen Type: Serum

Container/Tube: Red

Specimen Volume: 3 mL

Draw blood in a plain red-top tube. Maintain specimen at 37 degrees C until serum is separated from cells. Spin down and send 3 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

2 mL

Reject Due To

Hemolysis	Mild OK; Gross reject
Lipemia	Mild OK; Gross reject
Icterus	Mild OK; Gross reject
Other	Separator or gel tube

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Ambient	72 hours	

Clinical & Interpretive

Reference Values

Negative

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

1 to 7 days

Performing Laboratory Location

ARUP Laboratories

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

86940, 86941

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FDLS	Donath Landsteiner	18287-3

Result ID	Test Result Name	Result LOINC® Value
FDLS	Donath Landsteiner	18287-3