

## Overview

### Useful For

Aids in the diagnosis of epithelioid hemangioendotheliomas

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOA	IHC Additional, Tech Only	No	No
IHTOI	IHC Initial, Tech Only	No	No

### Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

### Method Name

Immunohistochemistry (IHC)

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required, order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green pathology address label and the pink Immunostains Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Specimen Type:** Tissue

**Supplies:** Immunostain Technical Only Envelope (T693)

**Container/Tube:** Immunostain Technical Only Envelope

# Test Definition: CAMTA

Calmodulin-Binding Transcription Activator 1  
(CAMTA1), Immunostain, Technical  
Component Only

**Preferred:**

Formalin-fixed, paraffin-embedded tissue block

OR

2 Unstained, positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick

**Acceptable:** None

**Digital Image Access**

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/ihc-stains/#FAQ>

**Forms**

If not ordering electronically, complete, print, and send an [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

**Reject Due To**

Wet/frozen tissue Cytology smears Non formalin fixed tissue Nonparaffin embedded tissue Non charged slides ProbeOn slides Snowcoat slides	Reject
---	--------

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

**Clinical & Interpretive**

---

**Clinical Information**

Calmodulin-binding transcription activator 1 (CAMTA1) protein overexpression in the nucleus is the result of the *WWTR1-CAMTA1* gene fusion, which is found in approximately 90% of epithelioid hemangioendotheliomas (EHE). CAMTA1 can be used to distinguish EHE from other tumors with epithelioid morphology.

**Interpretation**

This test does not include pathologist interpretation; only technical performance of the stain. If interpretation is required order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request; call 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

**Cautions**

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

The charge of glass slides can be affected by environmental factors and subsequently may alter slide staining. Sending unsuitable glass slides can result in inconsistent staining due to poor slide surface chemistry.

Best practices for storage of positively charged slides:

- Minimize time slides are stored after being unpackaged
- Limit exposure to high humidity and heat
- Minimize exposure to plastics

**Clinical Reference**

1. Shibuya R, Matsuyama A, Shiba E, et al: CAMTA1 is a useful immunohistochemical marker for diagnosing epithelioid haemangioendothelioma. *Histopathology*. 2015; 67:827-835. doi:10.1111/his.12713
2. Flucke U, Vogels RJ, de Saint Aubain Somerhausen N, et al. Epithelioid Hemangioendothelioma: clinicopathologic, immunohistochemical, and molecular genetic analysis of 39 cases. *Diagn Pathol*. 2014;9:131 doi:10.1186/1746-1596-9-131
3. Doyle LA, Fletcher CD, Hornick JL. Nuclear expression of CAMTA1 distinguishes epithelioid hemangioendothelioma from histologic mimics. *Am J Surg Pathol*. 2016;40:94-102. doi:10.1097/PAS.0000000000000511
4. Magaki S, Hojat SA, Wei B, So A, Yong WH. An introduction to the performance of immunohistochemistry. *Methods Mol Biol*. 2019;1897:289-298. doi:10.1007/978-1-4939-8935-5\_25

**Performance****Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

1 to 3 days

**Specimen Retention Time**

Until staining is complete.

**Performing Laboratory Location**

Rochester

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

**CPT Code Information**

88342-TC, primary

88341-TC, if additional IHC

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
CAMTA	CAMTA1 IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC® Value
603418	CAMTA1 IHC, Tech Only	Bill only; no result