

## Overview

### Useful For

p40 aids in the classification of carcinomas and lymphomas

Napsin A aids in the identification of primary lung adenocarcinoma

### Reflex Tests

| Test Id | Reporting Name           | Available Separately | Always Performed |
|---------|--------------------------|----------------------|------------------|
| IHMTO   | IHC Multiplex, Tech Only | No                   | No               |

### Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

### Method Name

Immunohistochemistry (IHC)

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Specimen Type:** Tissue

**Supplies:** Immunostain Technical Only Envelope (T693)

**Container/Tube:** Immunostain Technical Only Envelope

**Preferred:**

-Formalin-fixed, paraffin-embedded tissue block

OR

-2 Unstained, positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick

**Acceptable:** None

**Digital Image Access**

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/pathology/digital-imaging/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/pathology/digital-imaging/#section3>

**Forms**

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

**Reject Due To**

|   |        |
|---|--------|
| Wet/frozen tissue<br>Cytology smears<br>Nonformalin fixed tissue<br>Nonparaffin embedded tissue<br>Noncharged slides<br>ProbeOn slides<br>Snowcoat slides | Reject |
|---|--------|

**Specimen Stability Information**

| Specimen Type | Temperature         | Time | Special Container |
|---------------|---------------------|------|-------------------|
| TECHONLY      | Ambient (preferred) |      |                   |
|               | Refrigerated        |      |                   |

**Clinical & Interpretive**

**Clinical Information**

p40 is an antibody (detected by the chromogen 3,3'-diaminobenzidine) that recognizes the deltaNp63 isoform of p63. This isoform may exert an oncogenic effect and is selectively expressed in squamous cell carcinoma. Napsin A is an aspartic proteinase involved in the proteolytic processing of surfactant precursors in the normal alveolar epithelium. In normal tissues, napsin A is expressed in the cytoplasm of alveolar macrophages, type II pneumocytes, pancreatic ducts and acini, and in renal tubules (detected by the chromogen fast red). Napsin A has clinical utility for the identification of primary lung adenocarcinomas. Napsin A is also positive in a subset of thyroid and renal cell carcinomas (especially papillary types).

**Interpretation**

This test does not include pathologist interpretation, only technical performance of the stain. If interpretation is required, order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request; call 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

**Cautions**

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

The charge of glass slides can be affected by environmental factors and subsequently may alter slide staining. Sending unsuitable glass slides can result in inconsistent staining due to poor slide surface chemistry.

Best practices for storage of positively charged slides:

- Minimize time slides are stored after being unpackaged
- Limit exposure to high humidity and heat
- Minimize exposure to plastics

**Clinical Reference**

1. Whithaus K, Fukuoka J, Prihoda TJ, Jagirdar J. Evaluation of napsin A, cytokeratin 5/6, p63, and thyroid transcription factor 1 in adenocarcinoma versus squamous cell carcinoma of the lung. *Arch Pathol Lab Med.* 2012;136(2):155-162
2. Sterlacci W, Savic S, Schmid T, et al. Tissue-sparing application of the newly proposed IASLC/ATS/ERS classification of adenocarcinoma of the lung shows practical diagnostic and prognostic impact. *Am J Clin Pathol.* 2012;137(6):946-956
3. Nonaka D. A study of deltaNp63 expression in lung non-small cell carcinomas. *Am J Surg Pathol.* 2012;36(6):895-899
4. Kargi A, Gurel D, Tuna B. The diagnostic value of TTF-1, CK 5/6, and p63 immunostaining in classification of lung carcinomas. *Appl Immunohistochem Mol Morphol.* 2007;15(4):415-420
5. Magaki S, Hojat SA, Wei B, So A, Yong WH. An Introduction to the Performance of Immunohistochemistry. *Methods Mol Biol.* 2019;1897:289-298

**Performance**

**Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

**PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

1 to 3 days

**Specimen Retention Time**

Until staining is complete.

**Performing Laboratory Location**

Rochester

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

**CPT Code Information**

88344-TC

**LOINC® Information**

| Test ID | Test Order Name               | Order LOINC® Value   |
|---------|-------------------------------|----------------------|
| P40NA   | p40 + Napsin A IHC, Tech Only | Order only;no result |

  

| Result ID | Test Result Name              | Result LOINC® Value  |
|-----------|-------------------------------|----------------------|
| 70838     | p40 + Napsin A IHC, Tech Only | Bill only; no result |