

Overview

Useful For

Aiding in the identification of Wilms tumor

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOA	IHC Additional, Tech Only	No	No
IHTOI	IHC Initial, Tech Only	No	No

Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

Method Name

Immunohistochemistry (IHC)

NY State Available

Yes

Specimen

Specimen Type

TECHONLY

Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

Shipping Instructions

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

Specimen Required

Specimen Type: Tissue

Supplies: Immunostain Technical Only Envelope (T693)

Container/Tube: Immunostain Technical Only Envelope

Preferred:

-Formalin-fixed, paraffin-embedded tissue block

OR

-2 Unstained, positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick

Acceptable: None

Digital Image Access

1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: <https://news.mayocliniclabs.com/pathology/digital-imaging/>
2. Clients ordering stains using a manual requisition form will not have access to digital images.
3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: <https://news.mayocliniclabs.com/pathology/digital-imaging/#section3>

Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

Reject Due To

Wet/frozen tissue Cytology smears Nonformalin fixed tissue Nonparaffin embedded tissue Noncharged slides ProbeOn slides Snowcoat slides	Reject
---	--------

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

Clinical & Interpretive

Clinical Information

Wilms tumor-1 (WT-1) protein is a transcription factor that acts as a tumor suppressor gene. WT-1 is involved in

differentiation of certain tissues such as mesothelium and the urogenital system. It is also expressed in Wilms tumor, a kidney tumor found in children. In normal tissues, it is expressed in kidney cells, a subset of hematopoietic cells, Sertoli cells in the testis, granulosa cells in the ovary, and decidual cells of the uterus.

Interpretation

This test does not include pathologist interpretation, only technical performance of the stain. If interpretation is required, order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request; call 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Cautions

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

The charge of glass slides can be affected by environmental factors and subsequently may alter slide staining. Sending unsuitable glass slides can result in inconsistent staining due to poor slide surface chemistry.

Best practices for storage of positively charged slides:

- Minimize time slides are stored after being unpackaged
- Limit exposure to high humidity and heat
- Minimize exposure to plastics

Clinical Reference

1. Al-Hussaini M, Stockman A, Foster H, McCluggage WG. WT-1 assists in distinguishing ovarian from uterine serous carcinoma and in distinguishing between serous and endometrioid ovarian carcinoma. *Histopathology*. 2004;44:109-115
2. Hwang H, Quenneville L, Haziji H, Gown AM. Wilms tumor gene product: sensitive and contextually specific marker of serous carcinomas of ovarian surface epithelial origin. *Appl Immunohistochem Mol Morphol*. 2004;12:122-126
3. Lae ME, Roche PC, Jin L, et al. Desmoplastic small round cell tumor: a clinicopathologic, immunohistochemical, and molecular study of 32 tumors. *Am J Surg Pathol*. 2002;26(7):823-835
4. Nakatsuka S-I, Oji Y, Horiuchi T, et al: Immunohistochemical detection of WT1 protein in a variety of cancer cells. *Mod Pathol* 2006;19:804-814
5. Magaki S, Hojat SA, Wei B, So A, Yong WH. An introduction to the performance of immunohistochemistry. *Methods Mol Biol*. 2019;1897:289-298. doi:10.1007/978-1-4939-8935-5_25

Performance**Method Description**

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

1 to 3 days

Specimen Retention Time

Until staining is complete

Performing Laboratory Location

Rochester

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
WT1I	Wilm's Tumor (WT-1) IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC® Value
70894	Wilm's Tumor (WT-1) IHC, Tech Only	Bill only; no result