

Cocaine Analysis - Whole Blood

Overview

Reflex Tests

| Test Id | Reporting Name | Available Separately | Always Performed |
|---------|------------------------|----------------------|------------------|
| FCOKR | Cocaine, MS, WB/SP Rfx | No | No |

Testing Algorithm

If the Cocaine Analysis, whole blood is positive, then the Cocaine, Confirmation (FCOKR) will be performed at an additional charge.

Method Name

Immunoassay (IA)

Gas Chromatography with mass spectrometry (GC/MS); confirmation

NY State Available

Yes

Specimen

Specimen Type

Whole blood

Specimen Required

Draw blood in a purple-top (EDTA), green-top (sodium-heparin) or grey-top (NaF/oxalate) tube(s). Send 7 mL of whole blood refrigerated in a plastic vial.

Specimen Minimum Volume

3 mL

Reject Due To

| Hemolysis | NA |
|-----------|----|
| Lipemia | NA |
| Icterus | NA |
| Other | NA |

Specimen Stability Information

| Special container | Specimen Type | Temperature | Time | Special Container |
|-------------------|---------------|-------------|------|-------------------|
|-------------------|---------------|-------------|------|-------------------|



Test Definition: FCOKE

Cocaine Analysis - Whole Blood

| Whole blood | Ambient | 72 hours | |
|-------------|--------------------------|----------|--|
| | Refrigerated (preferred) | 14 days | |
| | Frozen | 180 days | |

Clinical & Interpretive

Reference Values

Cocaine Screen: Cocaine and Metabolite, UA – Negative; Cutoff: 25 ng/mL

Cocaine Confirmation: Cocaine Benzoylecgonine Confirmation threshold: 10 ng/mL

Performance

PDF Report

No

Day(s) Performed Monday through Sunday

Report Available 5 to 9 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact Customer Service.

Test Classification

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

CPT Code Information



Cocaine Analysis - Whole Blood

80307 80353-if applicable

LOINC[®] Information

| Test ID | Test Order Name | Order LOINC [®] Value |
|-----------|-------------------------|---------------------------------|
| FCOKE | Cocaine, Scr w/Conf, WB | 16633-0 |
| | | |
| Result ID | Test Result Name | Result LOINC [®] Value |
| | | |