

## Overview

### Method Name

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Serum Red

### Specimen Required

**Collection Container/Tube:** Red top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 3 mL

**Collection Instructions:** Draw blood in a plain, red-top tube(s), **serum-gel tube is not acceptable**. Centrifuge and send 3 mL of serum refrigerate in a plastic vial.

### Specimen Minimum Volume

0.5 mL

### Reject Due To

Hemolysis	Mild reject; Gross reject
Lipemia	Mild reject; Gross reject
Icterus	Mild reject; Gross reject
Other	Specimens other than Serum collected in red top. Plasma or whole blood collected in Sodium Heparin or EDTA. Serum gel tubes or Plasma gel are NOT acceptable.

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

## Clinical & Interpretive

**Reference Values**

Reference Range: 1.0 - 10 ng/mL

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Sunday

**Report Available**

5 to 9 days

**Specimen Retention Time**

2 weeks

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**CPT Code Information**

80342

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
PROLX	Fluphenazine (Prolixin)	3650-9

Result ID	Test Result Name	Result LOINC® Value
Z1122	Fluphenazine	3650-9