MAYO CLINIC LABORATORIES

TEST STATUS - TEST DOWN

Notification Date: December 14, 2023 Effective Date: Immediately

Clonazepam and 7-Aminoclonazepam, Serum

Test ID: CZPS

Explanation: This test will be made non-orderable effective immediately due to instrumentation issues and assay redevelopment. An update will be provided when testing resumes.

Recommended Alternative Test:

Clonazepam and Metabolite, Serum

Test ID: FCLMS

Methodology: High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

Reference Values:

Reporting limit(s) determined each analysis.

Clonazepam: None Detected ng/mL

7-Amino Clonazepam: None Detected ng/mL

Specimen Requirements:

Must submit one specimen per order. Specimens cannot be shared between multiple orders.

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic, preservative-free vial

Specimen Volume: 2 mL Minimum Volume: 0.7 mL Collection Instructions:

1. Draw blood in a red top tube(s). Serum gel tube is not acceptable.

2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

Specimen Stability Information:

Specimen Type	Temperature	Time
Serum Red	Refrigerate (preferred)	14 days
	Frozen	30 days

CPT Code:

80346

G0480 (if appropriate)

Miscellaneous Referral Test Fee: \$103.00

Day(s) Performed: Monday through Sunday Report Available: 7 to 11 days

Questions

Contact Brandon DeBoom, Laboratory Resource Coordinator at 800-533-1710.