



## THC Confirmation, MS, SP

**Explanation:** The following new orderable, referred to Medtox Laboratory, will be available on March 27, 2023.

**Test ID:** FHCCF

**Methods:**

Gas Chromatography-Mass Spectrometry (GC-MS)

**Reference Values:**

Negative

Units: ng/ml

**Specimen Requirements:**

Submit only 1 of the following specimens:

**Serum**

**Container/Tube:** Red Top

**Specimen Volume:** 5 mL

**Collection Instructions:** Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 5 mL of serum refrigerated in plastic vial.

**Minimum Volume:** 2 mL

**Plasma**

**Container/Tube:** Green-top (sodium heparin)

**Specimen Volume:** 5 mL

**Collection Instructions:** Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 5 mL sodium heparin plasma refrigerated in plastic vial.

**Minimum Volume:** 2 mL

**Specimen Stability Information:**

Specimen Type	Temperature	Time
Varies	Refrigerated (preferred)	14 days
	Frozen	180 days

**Fee:** \$163.42

**CPT Code:** 80349, G0480 (if appropriate)

**Day(s) Setup:** Monday through Sunday

**Report Available:** 7 to 9 days

**Questions**

Contact MCL Referrals Supervisor Amy Bluhm at 800-533-1710.