

Disaccharidase Activity Panel, Tissue

Test ID: DSAC

Explanation: On the effective date, supply item T993 (15 mL Tissue Tube-DSAC) will be implemented for specimen submissions. Tissue collection instructions have been updated.

Note: As clients implement the use of the T993 tube for DSAC tissue submissions, the laboratory will not cancel testing if tissue is received in an alternate container.

Current Specimen Required	New Specimen Required
<p>Specimen Type: Tissue Source: Intestinal biopsy Container/Tube: Clean, screw-topped plastic vial Specimen Volume: 5 mg Collection Instructions: Specimen should not be placed on gauze or filter paper, nor should any saline, support or embedding material be added.</p>	<p>Specimen Type: Tissue Source: Intestinal biopsy Supplies: 15 mL Tissue Tube-DSAC (T993) Container/Tube: 15 mL tissue tube Specimen Volume: 5 mg Collection Instructions: 1. Place specimen in tissue tube. 2. Specimen should not be placed on gauze, filter paper, or swabs/wooden sticks and should not have any saline, water, support, or embedding material added.</p>

Questions

Contact Melissa Tricker-Klar, Laboratory Resource Coordinator at 800-533-1710.