## **TEST OBSOLETE**



Notification Date: November 13, 2024 Effective Date: December 13, 2024

# Interleukin 1-Beta

Test ID: FINTB

### **Explanation:**

Due to changes in the laboratory information system by the performing laboratory, Inter Science Institute, a new test build is required. This test will become obsolete effective December 13, 2024, and the new recommended alternative test will be available and is listed below.

#### **Recommended Alternative Test:**

## Interleukin 1-Beta

Test ID: FIN1B

#### Methods:

Enzyme Immunoassay (EIA)

#### **Reference Values:**

Less than 1.0 pg/mL

#### **Specimen Requirements:**

Patient preparation: Patient should NOT be on any corticosteroids, anti-inflammatory medications, or pain killers, if possible, for at least 48 hours prior to specimen collection.

Specimen Type: Serum **Collection Container/Tube:** 

Preferred: Red top Acceptable: SST

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL **Collection Instructions:** 

- 1. Draw blood in a plain, red-top tube(s), serum-gel tube(s) is acceptable.
- 2. Centrifuge and aliquot 3 mL of serum into a plastic vial.
- 3. Send frozen

Minimum Volume: 1 mL

# **Specimen Stability Information:**

Specimen Type	Temperature	Time
Serum	Frozen	30 days

Fee: CPT Code:

\$260.00 83520

Day(s) Setup: Monday through Friday Report Available: 12 to 14 days

## **Questions**

Contact MCL Referrals Supervisor, Amy Bluhm, at 800-533-1710.