



Prostaglandin D2 (PG D2), Random Urine

Explanation: The following new orderable, referred to Inter Science Institute, will be available December 19, 2024.

Test ID: FPROS

Methods:

Direct ELISA

Reference Values:

Up to 175 ng/g Creatinine

Specimen Requirements:

Patient preparation: Patient should not be on aspirin, indomethacin, or anti-inflammatory medications, if possible, for at least 48 hours prior to collection of specimen.

Specimen Type: Urine

Submission Container/Tube: Plastic, 10-mL tube (T068)

Specimen Volume: 10 mL

Collection Instructions:

1. Collect 10 mL random urine (NO preservative).
2. Freeze immediately and send specimen frozen in plastic, 10-mL urine tube (T068)
3. Specimen must remain frozen prior to and during shipping.

Note: 24-hour urine collection is not acceptable.

Specimen Minimum Volume: 5 mL

Specimen Stability Information:

Specimen Type	Temperature	Time
Urine	Frozen	60 days

Fee:

\$305.00

CPT Code:

84150

Day(s) Setup:

Monday through Friday

Report Available:

5 to 9 days

Questions:

Contact MCL Referrals Supervisor, Amy Bluhm, at 800-533-1710.